



PRODUCT CODE: AC-FSHIELD **PRODUCT NAME:** FACE SHIELD **PRODUCT DESCRIPTION:** Full face shield with brow guard, adjustable head and crown band

PRODUCT COMPLIANCE

EN/ISO REQUIREMENT	<ul style="list-style-type: none"> DIN EN166:2002-4 Mechanical strength: protection against high speed particles B (120 M/S) Optical class: 1 Shade class: 2C-1,2
---------------------------	--

MATERIALS AND FEATURES

BROW GUARD	<ul style="list-style-type: none"> Flip-up brow guard with 5 point lens attachment on resting position. Locking ratchet adjustment system for secure and comfortable fitting. Has adjustable acetyl side tilt and engage screws.
CROWN STRAP	7 point adjustment crown strap to aid comfortable secure fit
VISOR	<ul style="list-style-type: none"> 23cm, 1mm transparent anti-fog polycarbonate visor which is optically clear and has no distortion. Wrap around shape provides 180 degree protection around the face area.

SUGGESTED ENVIRONMENTS

- Excellent protection against potential contamination from body fluids, blood pathogens or harmful chemical splashes which commonly occur in medical and healthcare environments.
- Works as an effective defence splash guard also suitable for pesticide spraying, dental work protection and in laboratories.
- Suitable for most typical industrial risk applications.

WARNINGS

- Protection is only given when the visor is correctly fitted and in the lowered position.
- Do not use in any electrical arc applications.
- Do not use for brazing, welding or high intensity light protection.
- Do not use for protection against laser or gamma light sources or in high temperature areas like furnaces.
- Do not use for recreational sporting activities like paint ball etc.
- Do not use if there are any visible scratches or cracks and appears damaged, rather replace.
- Do not do any modifications to the original design of the face shield or remove any of the original parts.
- Rinse and clean with mild soap and water solution and do not use solvents or chemicals to clean.
- Frequent use and direct solar radiation can accelerate the aging of the plastic components, if any aging is visible, face shield should not be used, and rather replaced.

OTHER INFORMATION

SIZING	One size fits all with adjustable head and crown straps for secure fit
PACKAGING	40 units per carton, brow guard and visor need to be assembled